

# The Perfect TV Chair

---

## OWNER'S MANUAL

---



The Best Power Lift and Recline Chair Made in the USA



*first***STREET**  
for Boomers and Beyond

**1-800-958-8324**  
1998 Ruffin Mill Road  
Colonial Heights, VA 23834  
[www.firststreetonline.com](http://www.firststreetonline.com)



Dear Valued Customer,

Thank you for purchasing The Perfect TV Chair, designed exclusively for *firstSTREET* by Golden Technologies. Your Perfect TV Chair is tested and certified by NRTL Met Labs to UL and CSA electrical standards to assure that you have purchased one of the safest chairs available. We believe that this product is made from the highest quality components available, is engineered to give you maximum service with minimum maintenance, and is backed up by the strongest warranty in the industry. We hope that you enjoy years of pleasurable use of your Perfect TV Chair and that this chair becomes a true assist to you in both sitting down and standing up.



Please take a few moments to review this manual to become familiar with the operation of this chair and our warranty. If you need additional information, please call *firstSTREET* toll free at 1-800-958-8324.

Also, please note the following safety information: This power lift and recline chair is a medical device designed to help you sit down and stand up. Do not allow children to operate the chair at any time. When not in use, the chair should be kept in the sitting position.

Again, thank you for selecting *firstSTREET* and The Perfect TV Chair.

Sincerely,

Robert Golden  
*Chairman and Founder*

	<table border="1"><tr><td>Complies with UL 73 and CSA C22.2 No. 68</td></tr></table>	Complies with UL 73 and CSA C22.2 No. 68	
Complies with UL 73 and CSA C22.2 No. 68			
Listing # E112417 FNAE 80-214A			

# Safety Guidelines

The symbols below are used in the owner's manual and/or on the product to identify warnings and notify you of important safety information. Make sure you read and understand them completely.



**WARNING!** Indicate(s) a hazardous situation which, if not avoided, could result in death or serious injury. This icon is represented as a black symbol on a yellow triangle with a black border.



**MANDATORY!** Failure to perform mandatory actions could cause personal injury and/or equipment damage. This icon is represented as a white symbol on a blue dot with a white border.



**PROHIBITED!** Indicate(s) an action(s) that should not be performed under any circumstance. Failure to adhere could cause personal injury and/or equipment damage. This icon is represented as a black symbol with a red circle and red slash.

## Safety Symbols

The symbols below identify warnings, mandatory actions, and prohibited actions. They can be found in the owner's manual and/or on the product. Make sure you understand all safety labels before operating the product. **Do not** remove safety labels from your product.



**Read and follow owner's manual!**



**Child and/or Pet Crush Hazard!**



**Pinch/Crush Point Hazard!**



**Transformer Crush Hazard!**




**Fire Hazard!**


*continued on next page*

continued from  
previous page

# Safety Guidelines

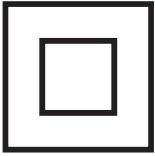

**Class II  
Device!**

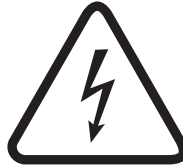

**Electrical  
hazard!**



**Indoor use  
only!**



**Do not dispose  
in trash!**



**Do Not  
Smoke!**



**Batteries  
required!**



## MANDATORY

**READ THIS MANUAL FIRST BEFORE INSTALLING OR OPERATING YOUR POWER LIFT AND RECLINE CHAIR.**

Your power lift and recline chair has been engineered from the leg levelers up to provide the ultimate in strength and reliability.

Great care has been taken to provide you with total comfort and safety. Golden Technologies thoroughly inspects and tests each unit prior to shipment.



## ! WARNING

Your Golden power lift chair is a medical device. Therefore, you are required to exercise caution when operating it to ensure your personal safety and that of others around you. The following are rules for the safe operation of your Golden power lift chair.



Care should be taken in the unpacking of this unit, so as not to cause injury to persons or damage to the unit. Dispose of all plastic bags and covers, as they can be a smothering hazard to small children.



Do not allow children or pets near the chair while the chair is operating.



Do not allow children to play with or operate the chair at any time.



Hand control should be unplugged and placed out of the reach of children when the chair is not in use.



Do not flop down into the chair when getting into it, or while it is in its lift position. Flopping down into the chair will damage it and could cause personal injury.



Do not allow anyone to sit on your lap, or on the arm of the chair, or on the footrest while in or operating the chair, as it may cause tipping and could result in personal injury.



Do not place fingers or hands between the seat cushion and the inside arm of the chair while sitting in the chair or when the chair is in operation or plugged into a power source.



Do not retrieve fallen objects from inside or under the chair while the chair is plugged into a power source.



Do not operate the chair if there are any persons, pets or other obstructions under or in front of the footrest, under the chair or behind the chair.



Do not smoke while operating, sitting or resting in the chair.



To reduce risk of electrical shock, do not immerse any of the electrical components into a liquid, or spill any liquids onto electrical components. Do not open any of the electrical components. There are no serviceable parts.



Warranty will be void if any of the electrical components are opened by other than a qualified service person.



Make sure the chair is placed at the specified distance from a wall/ corner (page 7). Failure to do so can damage the chair and will void the warranty.



Do not use any type of electrical extension cord to plug your chair into an electrical outlet. Make sure that all electrical products and their wires are positioned away from your chair as the metal lift frame mechanism could crush them and create a fire hazard and/or may cause personal injury.



Do not operate your lift chair while intoxicated or taking medications that impair your mental or physical abilities to operate the chair safely. Consult with your doctor about any medications you are taking that may affect the safe operation of the chair.



Do not operate this product if you have any medical conditions that result in limited, debilitating mental or physical capacity. You should consult your doctor before using this product.



Consult with your physician regarding using the Trendelenburg position.



The convenience table is not intended for the support of any person, adult or child. The weight limitation is 5 to 8 pounds. If exceeded, severe deflection will occur/or possible breakage.

---

## WARNINGS FOR TV TRANSMITTER



**WARNING** Before using the transmitter please read and understand this manual in its entirety.



### SAFETY WARNINGS

- Keep this product away from water, liquid, or wet areas.
- The unit should be powered off before plugging the product into the power outlet.
- This item is **ONLY** to be plugged into an AC 100V~240V power outlet.
- The unit should be powered off before unplugging the product.
- Do not place the product in direct sunlight or heat. Keep away from temperatures over 95°F/35°C.
- Do not use a wet cloth on the product. Always use a dry cloth to clean this product and keep it free from dust.
- Do not use chemicals on this product.
- Never try to repair this product by yourself, take it to an authorized technician.



Trying to remove panels or repairing the unit by yourself could result in an electric shock.

- Do not insert foreign objects into the product.
- For better reception, avoid walls and large objects between the transmitter and the Bluetooth receiving device.


---

## INSTRUCTIONS FOR THE CONVENIENCE TABLE


1. Open the right-side arm cushion.
2. Pull out the tray.
3. Lift the support arm up and out into an upright position. (The arm will fall into a semi-locked position preventing backward movement).
4. Close the arm cushion if desired.
5. Insert the tray onto one of the preferred hole locations.

## INSTRUCTIONS FOR THE PERFECT TV CHAIR

1. Remove the transformer from the transformer box (located behind the lower outside back) and install two 9 volt alkaline batteries into the sockets on the transformer. Place transformer on the floor behind the chair.
2. Plug the transformer into a 115-120V 60Hz A.C. outlet and connect the power cable from the chair into the transformer as shown. Power cable is located under the chair attached to the motor shaft with white zip tie. Remove zip tie, extend cable under and out behind the chair.

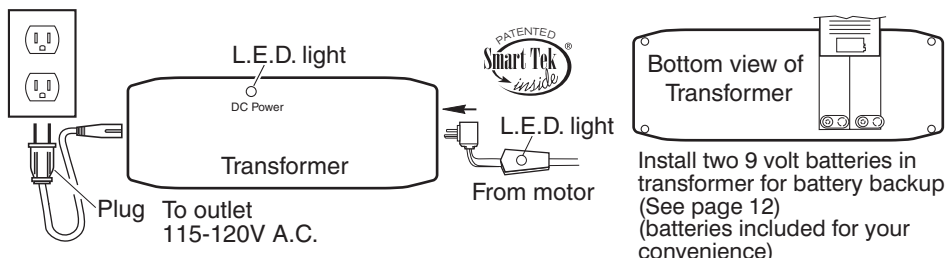

3. Place the chair in the desired position in the room, keeping it 26" away from the wall. If placing the chair into a corner, keep it 26" from the wall as shown.

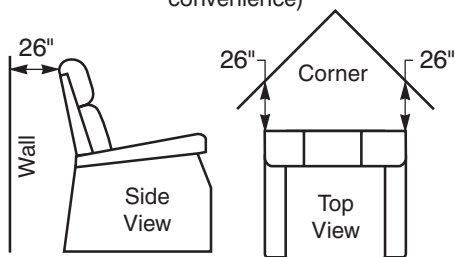

### Distance required from the wall when reclined:

Wall: 26 inches - Corner: 26 inches

4. Adjust the leg levelers on the bottom of the chair if necessary to level the chair. Some high pile carpets can obstruct the footrest operation by rubbing against it. Raise the levelers to a height above the pile.
5. Slide the hand control from between the back and arm out toward the back of the chair. Secure the cord into the quick release cable clamp located above the magazine pocket on the arm of the chair.


## AUTODRIVE HAND CONTROL OPERATION

1. **Getting into your lift chair.** While standing next to your chair, press button (S7) and raise the chair to a height you can sit against. This height will vary from person to person depending on the individual's needs. Raise the chair high enough so you do not have to flop down into the chair. **Flopping down into the chair will damage it and could cause personal injury.** Press button (S5). As the chair continues downward, adjust yourself into a position that is comfortable to you. Continue holding button until chair is in the seated position (chair firmly on the floor), then release the button. You can also press buttons (S6, S8, S9, S10) and the chair will move to those positions.

- 2. Getting out of your chair.** Press button (S7) and raise the chair to a height that enables you to easily stand. Release the button. Return the chair to the seated position (S5) (chair firmly on the floor) after getting out of the chair, so children, pets or obstructions do not get under the chair. Repeat no. 1 when you want to sit down.

## Micro Adjustments

- 1. To recline the back of the chair,** press button (S1) and hold until the back reaches the desired position. Then release the button.
- 2. To bring the back of the chair to an upright position,** press button (S2) and hold until the back reaches the desired position. Then release the button.
- 3. To raise the foot rest and seat,** press button (S3) and hold until the foot rest reaches the desired position. Then release the button.
- 4. To lower the foot rest and seat,** press button (S4) and hold until the footrest closes. Then release the button.

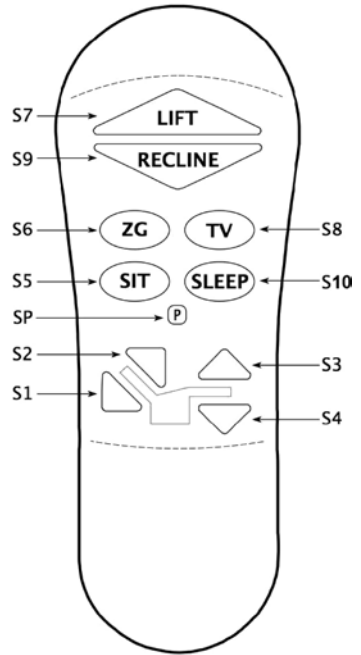

**Note:** The motors are set to shut off when they reach their fully extended or fully retracted position if the selected button is not released. If you should go past your desired position, select the button to bring you back to your desired position. You can repeat the process as often as you like.

## Programming the Memory Buttons S5, S6, S8 and S10

Press one of the preprogrammed buttons (S5, S6, S8, S10) and hold until the chair stops. Press buttons (S1) thru (S4) to micro adjust those positions to suit your needs. Use the following procedure to reprogram (S5, S6, S8, S10) buttons after you have the chair in your desired position.

1. Press and hold program button (SP). The back light goes off while pressed.
2. In about 3 seconds the backlight will start flashing (one flash)
3. Within 5 seconds, press one of the memory buttons (S5, S6, S8, S10) to save the position to the button pressed. The back light will return to non-flashing position indicating that the new position is stored.

If a programmable memory button is not pressed within 5 seconds, or a non-programmable button is pressed, then the programming sequence is aborted and you will need to repeat steps 1 to 3 above. You may reprogram the memory buttons as often as you like.



## Factory Reset

Resets all the programmed buttons (S5, S6, S8 and S10) back to the factory settings.

1. Press and hold program button (SP).  
The backlight goes off while button is pressed.
2. In about 4 seconds the backlight will start flashing.
3. Within 5 seconds, press button (S1).  
The backlight will change to non-flashing.
4. Within another 5 seconds, press button (S4). Factory reset is done.

If button (S1) is not pressed within 5 seconds, or a different button is pressed, then the programming sequence is aborted and you'll need to repeat steps 1 to 4 above.

## Function Lock

Should you ever want to limit the button function of your hand control, you can use the function lock feature to disable all button function except for buttons (S7 and S9).

1. To lock function, press and hold the program button (SP) for 4 seconds until the backlight starts flashing.
2. Press button (S1), press button (S2), press button (S3), press button (S6).
3. To unlock function, press and hold the program button (SP) for 4 seconds until the backlight starts flashing.
4. Press button (S1), press button (S2), press button (S3), press button (S6).


## Hand control disable

Hand control disable, disables all functions on the hand control

1. To disable, press the program button (SP) and then press button (S7).
2. To enable, press the program button (SP) and then press button (S7).

## PERFECT TV CHAIR POSITIONS


Your Perfect TV Chair has been equipped with the MaxiComfort™ System. The MaxiComfort System has been designed specifically for you. Offering you a wide range of body positioning, so you can choose the position that best suits your needs. The MaxiComfort System accomplishes this by providing four primary motions: three recline, one lift and combinations of those motions, allowing you to settle into your favorite comfort position. Shown below are some of the MaxiComfort positions. There are many to choose from. You may stop your Perfect TV Chair at one of the positions shown or anywhere in between just by releasing the buttons on the hand control when the chair reaches the position you desire

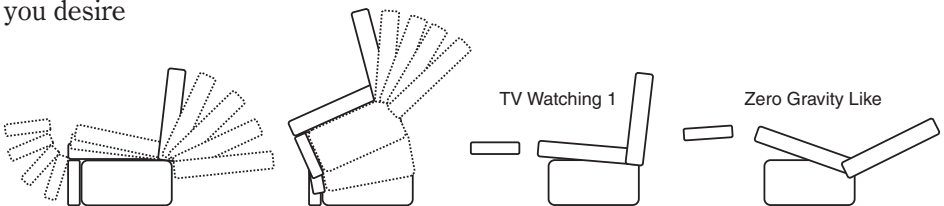

Drawings are for illustration purpose only

# CONTENTS INCLUDED IN THE AUDIO TRANSMISSION SYSTEM

No.	Name	Specification
1	TV Audio Transmitter	L 3 15/16" x W 2 1/8" x H 3/8"
2	Audio Input Cable	39 3/8" + 7 7/8" x 5/64"
3	Power Adaptor	5V/1AMP

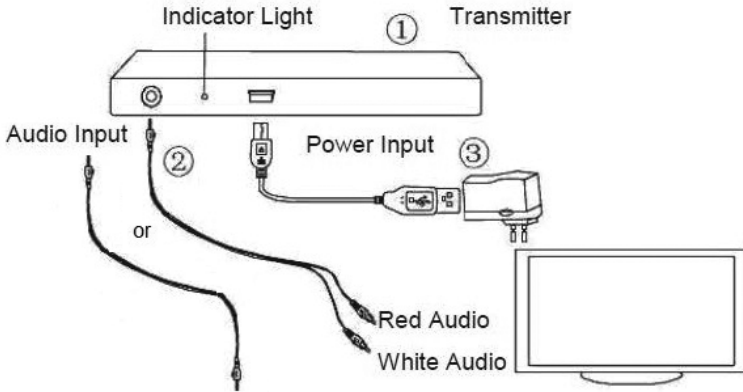

The Perfect TV Chair allows the owner to connect the audio output from a television and have it play through the built in speakers in the Perfect TV Chair. These instructions should be followed to get you started the first time.

## Initial Set Up


1. Identify the components included to connect to your TV from the list above.
2. Connect the transmitter to the Audio Out jacks on your TV. The end of the cable with two male ends connect to the Red/White Audio Output connections on your TV (generally located in the jack panel on the back of the TV).
3. Connect the other end of the Audio Input Cable to the appropriate connection on the TV Audio Transmitter.
4. Connect the Power Adaptor to an AC Power wall outlet and to the TV Audio Transmitter (in the appropriate connection on the back of the transmitter).
5. Power up the audio system in the Perfect TV Chair by pressing the Power Button on the control panel hidden in the left arm of the chair. Also depress the "Mode" button (repeatedly as necessary) until the indicator light turns blue. It may take up to a minute for this connection to occur the first time the entire system is set up.
6. Listen through the speakers for a series of beeps that indicate the Transmitter and the Perfect TV Chair are connected. Adjust the volume as necessary to hear the beeps.
7. For best results, please ensure that the Perfect TV Chair is no further away than 32 feet unobstructed from the TV and the TV Audio Transmitter.

## To Use the TV Sound System

1. Depress the power button on the control panel hidden in the left arm of the chair.
2. Depress the Mode button until the indicator light glows blue.
3. Adjust the TV settings to enable "External Speakers". This setting is generally found in the Audio section in the Menus on your TV.

## Troubleshooting Tips

Problem	Possible Cause and Correction
Unit does not turn on.	AC adaptor isn't plugged into a working wall outlet and/or into the power supply jack properly.
Unit does not transmit.	<ol style="list-style-type: none"> <li>1. Make sure the red and blue indicator lights on the chair control panel are illuminated.</li> <li>2. Turn off the power on the chair control panel and power back on in 3 seconds.</li> <li>3. If you have other Bluetooth devices in the same area, turn them off.</li> <li>4. If there is no audio connection after 3 minutes, power off the transmitter and control panel, power both on and retry setup.</li> <li>5. The Perfect TV Chair speakers will not work if the headphones are connected to the headphone jack.</li> </ol>
Blue indicator light is on but there is poor audio or no audio at all.	<ol style="list-style-type: none"> <li>1. Check audio connection on the transmitter.</li> <li>2. Check audio connections on the TV.</li> <li>3. Check for obstructions between the TV and the Perfect TV Chair.</li> <li>4. Make sure the effective transmitting distance of 32 feet is not exceeded.</li> </ol>

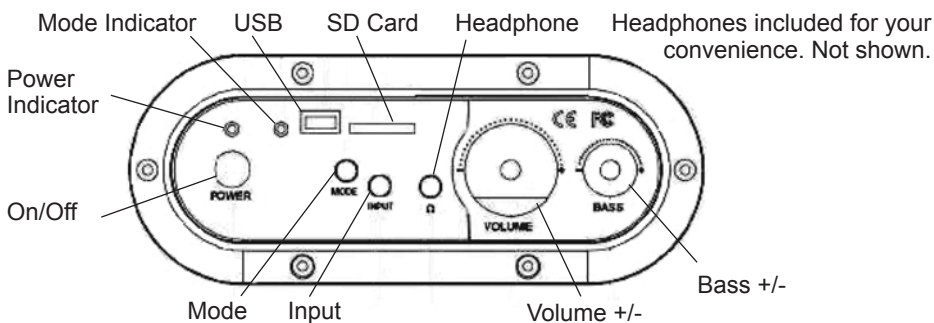

**POWER:** Press to turn on or off.

**INPUT:** Plug the AUX cable into the INPUT to play music from your MP3 / MP4 / pad...etc.

**MODE:** Press to set mode.

**MODE LIGHT:** Blue light indicates TV mode; Green light indicates USB / SD card mode; light off indicates AUX mode.

**BASS:** Increase / decrease subwoofer volume.

**VOLUME:** Increase / decrease speaker volume.

**HEADPHONE:** Use for headset in any mode.

**USB:** Use to charge your mobile phone. Rating: 5 volt / 0.5 AMP

## Battery Back Up System

Your Perfect TV power lift chair comes with a battery back up system so the chair can return to the seated position in the event of a household power failure. A fresh set of standard 9V batteries have enough energy to bring the chair to a seated position from either recline or lift. *The batteries do not have enough energy nor are they intended to operate the chair for a prolonged period of time in the absence of standard household voltage (120 volt, 60 Hz).* **Batteries should be replaced after a single use.** Batteries should be changed semi-annually (such as with smoke alarm batteries). For your convenience, an initial set of batteries is provided for your use. It is the user's responsibility to ensure that fresh batteries are used in the battery back up system. To help ensure the longest life of the batteries, users are encouraged to not allow the remotes to become wedged in between the seat cushions and the chair as this may activate the backlights on the remote and create a slow drain of the batteries. (See installation page 7.)

### Smart Tek™ (Power Indication)

Your power lift chair is equipped with an indicator light on the hand control. This light illuminates green when any button on the hand control is pressed indicating power to the hand control.



Your power lift chair also comes with a power indicator light on the transformer and on the power cord connecting the motor to the transformer. The lights illuminate green constantly, indicating power from the outlet to the transformer and the motor. The lights have been provided to help you identify electrical disconnects, should they occur, and to help your provider to diagnose electrical problems, should they occur.

Should the chair not operate and the lights are out, check all electrical connections to the chair. Make sure everything is plugged in properly and there is power to the outlet into which the chair is plugged (see installation on page 7 for electrical connections). **Should you ever encounter a problem with the operation of your power lift chair and the lights are on, such as mechanical problems or one light is out, discontinue all use of the product upon discovery of the problem.** Contact the provider from whom you purchased your chair. They will be able to resolve problems and take care of proper factory service if needed. You will need to be prepared to answer some basic questions about your power lift chair, such as model and serial number. The model and serial number are located on the hand control cord and the rear of the lift frame. Your provider will ask you questions about your power indicator lights, such as which one is out. Prior to calling your provider, check all the electrical connections to the chair. Make sure everything is plugged in properly and there is power to the outlet into which the chair is plugged (see installation on page 7 for electrical connections).



# GOLDEN PRODUCT REGISTRATION

Register Now To Lock In Your Warranty and Receive Important Product Updates

**SAVE TIME - REGISTER ONLINE: [WWW.GOLDENTECH.COM](http://WWW.GOLDENTECH.COM)**

## SERIAL NUMBER

Most Lift Chairs Have On Controller Wire or Back Rail of the Metal Frame  
Most Scooters & Powerchairs On Seat Post, Under/On Battery, or Under Carpet

## PRODUCT MODEL NUMBER

Most Lift Chairs Start with PR (PR401, PR510, etc).  
Most Scooters with G (GA, GB, GC, GL, GR, etc) Most Powerchairs with GP

## PURCHASE DATE

## FIRST NAME

## LAST NAME

## ADDRESS

## ADDRESS LINE 2

## CITY

## STATE / PROVINCE ZIP / POSTAL CODE

## COUNTRY

## PROVIDER NAME

## PROVIDER'S ADDRESS

## ADDRESS LINE 2

## CITY

## STATE / PROVINCE

## ZIP / POSTAL CODE

## COUNTRY

Product comments:

Rate Provider Experience (Circle One - 1 Bad, 5 Great)

**1 2 3 4 5**



PLACE  
STAMP  
HERE



GOLDEN TECHNOLOGIES  
401 BRIDGE ST  
OLD FORGE PA 18518-2323



# CONSUMER LIMITED LIFETIME AND SEVEN YEAR WARRANTY PERFECT TV CHAIR MODELS

**GOLDEN**, through its authorized distributor/service organization, warrants this chair to original consumer only, against defects in material or workmanship as follows:

## LIFETIME WARRANTY

**GOLDEN**, warrants for the lifetime of the chair by the original consumer to repair or replace at their option the Lift Mechanism, Chair Frame and Recline Mechanism. Labor to replace these parts is subsidized on all models for three years.

**GOLDEN**, shall not be responsible or obligated to pay for any cost or expense for examination of any warranted parts nor is Golden to pay for costs for transportation, handling and packaging of the parts or components.

## THREE YEAR PARTS WARRANTY

**GOLDEN**, warrants the electrical parts of this chair against defects to original consumer only, for a period of three years from the date of original purchase by a consumer. Labor to replace these parts is subsidized for one year on all models.

**GOLDEN**, shall not be responsible or obligated to pay for any costs or expenses for examination of any warranted parts nor is Golden to pay cost for transportation, handling and packaging of the parts or components.

## ONE YEAR PARTS WARRANTY

**GOLDEN**, warrants the TV/Audio system of this chair against defects to original consumer only, for a period of one year from the date of original purchase by a consumer. Labor to replace these parts is subsidized for one year on all models.

**GOLDEN**, shall not be responsible or obligated to pay for any costs or expenses for examination of any warranted parts nor is Golden to pay cost for transportation, handling and packaging of the parts or components.

## LIMITED WARRANTY FOURTH THROUGH SEVENTH YEAR

During the period beginning with the expiration of the full three year parts warranty offered above and continuing through the seventh year after date of original purchase by a consumer, **GOLDEN** will warrant the electrical parts on the following pro-rated basis. Year 4 earns a 35% discount off suggested retail price. Year 5 earns a 25% discount, year 6 a 15% discount and year 7 a 5% discount. By way of example, a part costing \$100.00 that needs replacement in year 4 would cost \$65.00.

**GOLDEN** shall not be responsible or obligated to pay for any costs or expense for examination and/or service of any warranted part or to pay for costs of transportation, handling and packaging of the chair and/or parts or components.

## TERMS APPLICABLE TO WARRANTY

**GOLDEN** will, at its option, repair and/or replace the defective parts and/or components provided that the warranty parts have not been subject to misuse, abuse or improper service by the consumer, have not been damaged in transit or handling and/or have not been altered or repaired by unauthorized representatives. This warranty does not include costs for unnecessary service calls, i. e., in-home service calls solely for the purpose of educating the consumer about the product, or finding an unsatisfactory power connection, etc. Exceeding the weight capacity of the chair will void the warranty

The Perfect TV Chair weight capacity is 375 lbs.

## WARRANTY EXCLUSION

**GOLDEN**, does not provide warranty for the batteries. Batteries are a consumable item that can drain, even without use, over time.

**In the event that the product does not conform to this warranty at any time while this warranty is in effect, Golden will correct the defect in conformity with the Consumer Limited Lifetime and Seven Year Warranty. THE LIMITED LIFETIME AND SEVEN YEAR WARRANTY SET FORTH ABOVE IS THE SOLE AND ENTIRE WARRANTY PERTAINING TO THE PRODUCT AND IS IN LIEU OF AND EXCLUDES ALL OTHER WARRANTIES OF ANY NATURE WHATSOEVER, WHETHER EXPRESSED, IMPLIED, OR ARISING BY OPERATION OF LAW INCLUDING, BUT NOT LIMITED TO ANY IMPLIED WARRANTIES OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE. THIS WARRANTY DOES NOT COVER OR PROVIDE FOR THE REIMBURSEMENT OR PAYMENT OF INCIDENTAL OR CONSEQUENTIAL DAMAGES.**

**Some states do not allow this exclusion or limitation of incidental damages so the above limitation or exclusion may not apply to you.**

**GOLDEN** shall not be liable for any incidental or consequential damage or loss not contained in this Warranty.

This warranty gives you specific legal rights, and you may also have other rights, which may vary from state to state.

All warranty claims must be approved by Golden Technologies before any service is performed.

Warranty is valid in all 50 states, Canadian Provinces and Puerto Rico.

# FOR YOUR RECORDS

Please fill in your Power Lift & Recline Chair information below. This information will be useful in the event that you should ever need to contact Golden Technologies, Inc. concerning your lift chair.

Date of Purchase \_\_\_\_\_

Model \_\_\_\_\_

Serial Number \_\_\_\_\_

Your Golden Technologies, Inc. provider.

*first*STREET  
for Boomers and Beyond

1998 Ruffin Mill Road  
Colonial Heights, VA 23834

## CARE AND MAINTENANCE

Your lift chair will require routine maintenance checks. You can perform some of these checks, while others may require assistance from a provider. By following these maintenance checks, you can make sure your lift chair gives you years of trouble-free operation. If you have any questions or concerns regarding your lift chair's care and maintenance, contact your provider.

- ✓ Before cleaning your lift chair, consult your provider for the proper types of cleaner and cleaning methods.
- ✓ Keep all electrical components free from moisture and liquid spills. **Intended for indoor use only.**
- ✓ Periodically inspect all wiring harnesses to make sure they are not cut, damaged, frayed, or disconnected. If damage of any type is present, unplug the lift chair and contact your provider.
- ✓ Periodically check all fasteners for tightness, wear, and corrosion. If any of these are present, unplug the lift chair and contact your provider.



**Warning:** Be careful of pinch points and sharp metal parts (wear gloves).

- ✓ Replace the batteries after one use. Replace the batteries yearly, even if they were not used. **Batteries are intended for one use only!**



### WARNING



Do not allow children to play on this mechanized furniture or operate the mechanism.

Leg rest folds down on closing so that a child could possibly be injured.

Always leave chair in a sitting position. Keep hands and feet clear of mechanism. Only the occupant should operate it.