


# AIRIAL CHAMBER

## HOLDING CHAMBER FOR METER DOSE INHALERS


Model # MQ8000

1

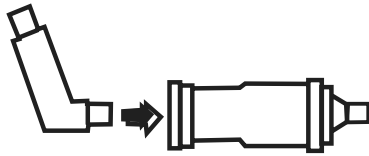


Remove MDI canister mouthpiece cover. Insert MDI canister mouthpiece into the elastic opening at the back of the AIRIAL CHAMBER.

2

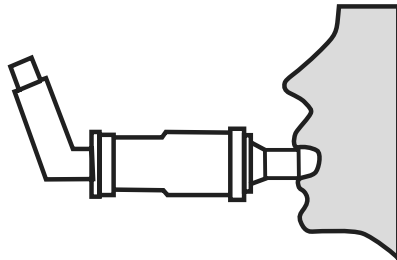


Insert CHAMBER mouthpiece between teeth, seal lips around mouthpiece.

3

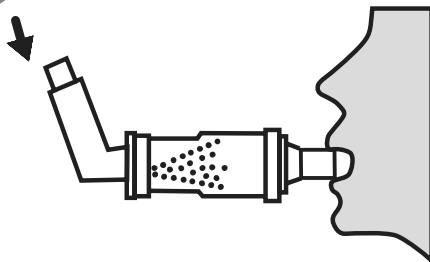

Sit up straight and exhale. Release one puff of MDI Medication into CHAMBER.

4

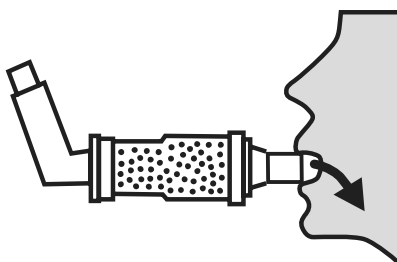

Take a slow, deep breath through the mouthpiece. Hold breath for 5-10 seconds.


**drive**<sup>TM</sup>

# AEROCÁMARA AIRIAL

## AEROCÁMARA PARA INHALADORES DE MEDICAMENTO


Artículo # MQ8000

1

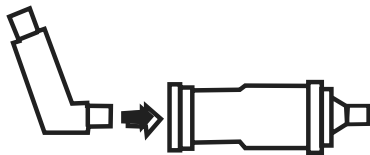


Retire MDI cubierta de la boquilla  
bote. Inserte MDI bote boquilla  
en la abertura elástica en la parte  
posterior de la AIRIAL CÁMARA.

2

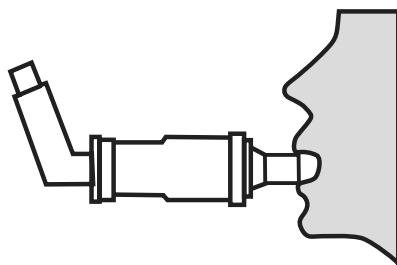


Inserte la boquilla CÁMARA entre  
los dientes, cerrar los labios  
alrededor de la boquilla.

3

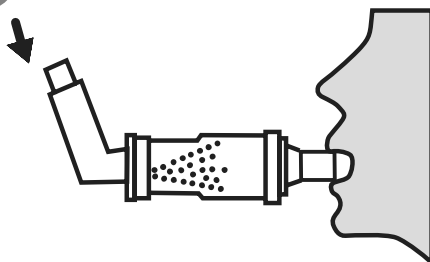

Siéntese derecho y exhala. Suelte  
una bocanada de MDI medicación  
dentro de la cámara.

4

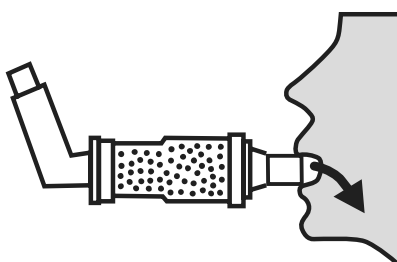

Tome una respiración lenta y  
profunda a través de la boquilla.  
Mantenga la respiración durante 5  
a 10 segundos.